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Safety Information for Nano Campo Control Unit with Mat (EN)

NanoCampo Control Unit with Mat, Charging Cable

Trade Name: NanoCampo
S/N: 202403

Technical Specifications and Certifications (Declaration of Conformity)

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Technical Specifications and Certifications (Declaration of Conformity)

- Maximum Operating Altitude: 2000 m
 - Charging Temperature: 10-40°C
 - CE Certification: This device complies with EU directives and Swiss standards.
 - Battery Information: Rechargeable LiPo Battery TYPE 906090, Rating: 3.7V, 6000mAh, 22.2Wh
 - AC/DC Charger: 230V ~50Hz, 15.0W - 5V 3A
- Use only with AQ15E-050A power supply unit.

Reference to Other Documents

This manual includes references to other relevant documents, standards, or websites.

Read the manual

Information on the functionality and application of the mat, as well as instructions for operating the software, can be found in the manual!

<https://nanocampo.ch/handbuch>

General Safety Instructions

General Notes

- Read this manual carefully before using the device.
- Keep this manual for future use.
- Follow all safety instructions to avoid injury and damage.

Scope of Application

- Recommended Uses: Description of suitable applications for the product (e.g., home use, therapeutic applications).
- Not Recommended Uses: Indications of what the device should not be used for (industrial applications, medical applications, medical emergencies). See also the section „The NanoCampo Mat is not intended for medical purposes.“

Safety Instructions

- This device is not intended for use by persons

(including children) with reduced physical, sensory, or mental capabilities or lack of experience and knowledge unless they are supervised or instructed on how to use the device by a person responsible for their safety. Children should be supervised to ensure they do not play with the device.

- Devices must only be powered by SELV.
- Use the device only as described in this manual.
- Connect the device only to a suitable power source that meets the requirements specified on the nameplate.
- Do not use the device if it shows visible damage or if the power cord is damaged.
- Do not leave the device unattended when in use.
- Unplug the device when not in use or before cleaning.

Product Description and Composition

- The NanoCampo Control Unit features connection cables, a power adapter, and an integrated battery. The mat consists of an outer layer of synthetic leather and contains metallic parts.

Implementation, Installation, Adjustments

- Ensure the control unit is placed on a stable and level surface.
- Connect the control unit to the power adapter and plug it into a suitable socket.
- Fully charge the integrated battery before using the device for the first time.
- Use only the supplied connection cables and accessories.

Operation of the Product

- Refer to the separate manual for software and operation of the programs and the device.
- Place the mat on a clean and dry surface.
- Ensure that the outer layer of the mat, made of synthetic leather, is intact and not damaged.
- Avoid contact of the mat's metallic parts with liquids.
- Do not place heavy objects on the mat to avoid damage.
- Do not fold or roll the mat too tightly to avoid breaking the wires.

Warning!

Do not fold or roll the mat too tightly:

The NanoCampo Mat should not be folded or rolled too tightly, as the wires inside may break.

- Ensure the mat does not overheat. Make sure there is adequate ventilation.
- Do not drop the control unit or subject it to shocks or high pressure (NanoCampo Control Box). The casing can break under excessive load.
- Maintenance and Cleaning
- Turn off the device and unplug it before cleaning.
- Clean the device and the mat with a soft, dry cloth.
- The mat consists of synthetic leather; clean the surface of the mat with a soft, dry cloth.
- For heavy soiling, use a slightly dampened cloth.

Ensure the surface is not soaked. Wipe the mat with a dry cloth after cleaning.

- Do not use harsh cleaning agents or solvents.
- Ensure that no liquids enter the device.
- Regular care: Clean the synthetic leather surface regularly to prevent dirt and wear and to ensure the mat's longevity.

Storage and Transport

- Store the device and mat in a dry and cool place.
- Protect the device and mat from direct sunlight and extreme temperatures.

Troubleshooting and Repair

- The battery should only be replaced by authorized personnel. Return the device to the manufacturer for repair.
- In case of gas/liquid leakage from the battery: Turn off the device immediately.
- Disconnect the device from the power source.
- Contact the manufacturer's customer service.
- Avoid contact with the leaked substance and clean the affected area with a dry cloth.
- Accessories and Peripherals
- Use only accessories and peripherals recommended by the manufacturer.

Disassembly and Environmental Disposal

- Dispose of the device and mat in an environmentally friendly manner and according to local regulations.
- Electrical devices and batteries must not be disposed of with household waste.
- Return the device to the manufacturer for disassembly and environmentally friendly disposal.

Warranty Conditions

- Details about the warranty, including duration, coverage, and exclusions, can be found on the website nanocampo.ch.

Support or FAQ

- Further information and FAQs, as well as support options, can be found on our website nanocampo.ch.

Manufacturer's Contact Information

- For questions or issues, please contact the manufacturer's customer service. info@nanocampo.ch

The NanoCampo Mat is NOT intended for medical purposes.

Important Safety Information, Health Advice and Precautions

- Consultation for Health Issues: People with health problems should consult a doctor before using the mat. This is especially important for individuals with severe or chronic conditions.
- Use During Pregnancy: Avoid using the mat during pregnancy.
- Therapeutic Application: NanoCampo mats are used to accompany various ailments but do not replace medical treatment and advice. In private settings, these mats are also used as relaxation

and wellness devices to enhance general well-being.

Scientific Evidence and Personal Experiences

- **Scientific Status:** The health benefits of these mats are not proven by current scientific standards.
- **Personal Experiences:** All statements in this manual are based on personal experiences and practical applications.
- **No Substitute for Medical Advice:** The NanoCampo NanoMagnetic Field Therapy is not a substitute for medical advice and care. Many conditions still require medical supervision. The experiences and values presented here do not reflect common medical opinion.

Warning! Not a substitute for conventional medical treatment:

The NanoCampo NanoMagnetic Field Therapy is not a replacement for conventional medical advice, treatment, and care.

Consultation for health problems:
Individuals with health issues should consult a doctor before using the mat.

Disclaimer and Personal Responsibility

- **User Responsibility:** Anyone using the information from this manual does so at their own risk. No diagnoses, therapy recommendations, or healing promises are made.
- **No Medical Advice:** All content is for information and self-help purposes only and does not replace a doctor's visit. Discuss any uncertainties with your doctor.

Special Groups

- **Infants and Pregnant Women:** Infants and pregnant women should not use magnetic field mats as potential risks are not sufficiently researched.
- **Medical Recognition:** The effectiveness of these mats is not scientifically proven.
- **Insurance Coverage:** Health insurance companies do not cover the cost of magnetic field mats.

Supplementary Notes

- **Unscientific Statements:** All application recommendations are based solely on empirical experiences and user statements. They have no scientifically proven background.
- **Combination with Other Therapies:** NanoCampo is a complementary health concept and can be combined with medical and alternative medical therapies. It is not a healing method and makes no healing promises.
- **No Substitute for Medical Therapy:** The NanoCampo concept does not replace medical therapy or advice.

Summary

The NanoCampo Mat is intended to promote general well-being and support regeneration, but it is

not designed for medical applications. Always consult a doctor before use, especially if you have health concerns or are pregnant. All provided information is based on personal experiences and is not scientifically proven.

Disclaimer

Exclusions and Disclaimers by the Manufacturer

General Disclaimers

- The manufacturer is not liable for damages resulting from improper use of the NanoCampo Mat.
- The manufacturer is not liable for injuries or health damages arising from using the mat without prior medical consultation, especially for individuals with health problems or during pregnancy.

Application Areas and Usage Conditions

- The NanoCampo Mat is not intended for medical purposes and does not replace medical treatment.
- The use of the mat is at the user's own risk. No diagnoses, therapy recommendations, or healing promises are made.
- The manufacturer is not liable for the effectiveness of the mat concerning health benefits, as these are not scientifically proven.

Warranty Limitations

- The manufacturer's warranty covers only material and manufacturing defects under normal use conditions.
- Damages caused by improper handling, misuse, modifications, or repairs by unauthorized persons are excluded from the warranty.
- The manufacturer is not liable for consequential or indirect damages resulting from the use of the mat.

Usage Notes and Personal Responsibility

- It is the user's responsibility to ensure that the mat is used according to the instructions in this manual.
- The manufacturer is not liable for damages resulting from ignoring safety and usage instructions.
- All usage recommendations are based on empirical experiences and serve informational purposes only. They do not replace professional medical advice.

Legal Rights

- These disclaimers do not affect the consumer's statutory rights under applicable consumer protection laws.
- By using the NanoCampo Mat, you acknowledge that you have understood and accepted the manufacturer's disclaimers. For questions or concerns regarding the use of the mat, contact the manufacturer's customer service or consult a doctor.

Manual and Instructions
For detailed information on how to use your NanoCampo Mat, please download the manual:

www.nanocampo.ch/handbuch

Manufacturers and Distributors Switzerland

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Mehrwertsteuernummer
MwSt.-Nr. CH-115.875.315

Distributors outside of Switzerland

FormSlim UG ((Limited Liability) DE & EU
Daniel Hertig
Rathausplatz 21 D-87435 Kempten

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Gesundheitszentrum Würenlos, Haselhalde GmbH Schweiz
Haselhalde GmbH